Fresenius Medical Care

DIASAFE[®] *plus*


Dialysis Fluid Filter

CE 0123

Technical Data

Filtration rate	≥ 300 mL/h·mmHg ≥ 3.75 L/min·bar
Maximum filtration pressure	2 bar
Membrane material Housing material Potting material Sealings Connection	Fresenius Polysulfone [®] Polypropylene Polyurethane Silicone DIAFIX [™] Lock System
Surface area	2.2 m ²
Endotoxin retention	$\geq 10^6$
Period of use / filter exchange	max. 12 weeks or 100 ONLINE <i>plus</i> treatments or if integrity test fails
Art. No.	5008201

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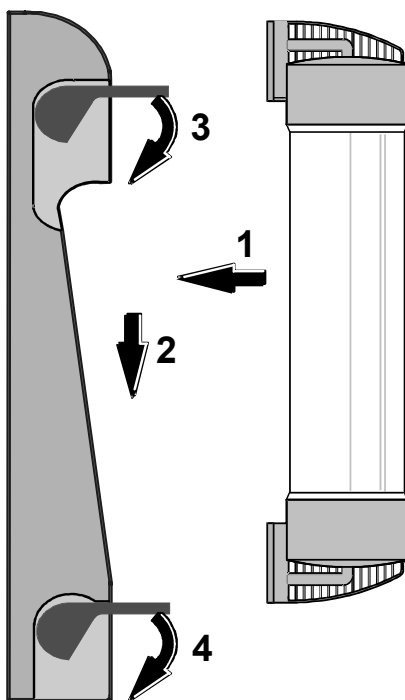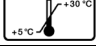
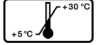
GB Instructions for Use

DIASAFE[®] plus Dialysis Fluid Filter

GENERAL INFORMATION

Refer to product or carton label for:

	Expiry date		Refer to instructions for use
	Batch		Order number
	Date of manufacture		Storage temperature range
	Units		

Indication: The DIASAFE[®] plus is intended for the preparation of ultrapure fluid for use in dialysis. Attention must still be given to the chemical and microbiological quality of water and concentrates and the maintenance of supply systems (e.g. RO system, central delivery system).

Chemical quality must comply with the European Pharmacopoeia. The microbiological quality (microbial count [CFU/mL] and endotoxin measurement [EU/mL]) should be:

Water: < 100 CFU/mL and < 0.25 EU/mL
Dialysis Concentrate: < 1000 CFU/mL and < 1 EU/mL

WARNINGS

The DIASAFE[®] plus can only be used with Fresenius Medical Care dialysis machines fitted serially for use with DIASAFE[®] plus or appropriately modified by authorised party. Attention must be given to disinfection and maintenance of all equipment. Please refer to the Operating Instructions of the dialysis machine for further details.

Disinfection / Cleaning

DIASAFE[®] plus is included in all machine cleaning and disinfection cycles. Disinfection according to the operating instructions is essential. Please refer to the Operating Instructions of the dialysis machine for further details.

Filter Integrity

When operated with a 4008, 5008 series machine, the integrity of DIASAFE[®] plus is automatically tested. The mandatory filter integrity test must be performed manually when operated with 2008 series machines in accordance with the relevant Operating Instructions.

Filter Life-Time

DIASAFE[®] plus must be exchanged after 12 weeks or 100 treatments (ONLINE plus) or earlier if the filter fails the integrity test. Please refer to the Operating Instructions for further details.

Installation / Exchange

DIASAFE[®] plus must only be installed in the specially designed filter holder of the dialysis machine. The DIAFIX[™] Lock System ensures the safe connection between the filter and the hydraulic components of the machine. Following correct installation disinfection is mandatory. Please refer to the Operating Instructions for further details.

WARRANTY

Products with manufacturing defects will be replaced if the defect is reported with details of the lot number.

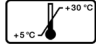
The manufacturer shall not be liable for any misuse, improper handling, non compliance with instructions for use and cautionary notes and for any damage incurred subsequent to the manufacturer's delivery of the DIASAFE[®] plus.

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Fresenius Medical Care AG & Co. KGaA